


Aseega Home

Sleep Diagnostic-Aid Software

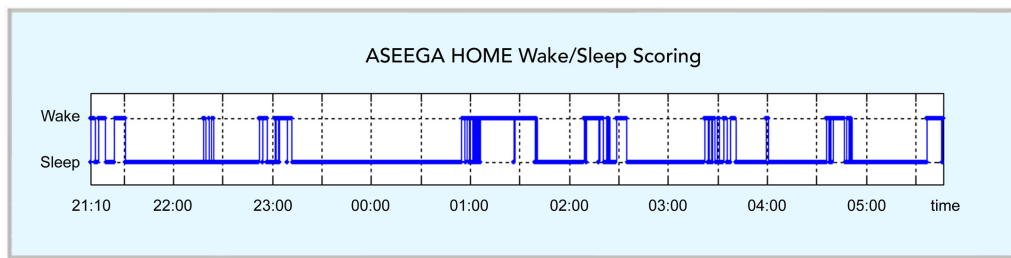

AHI enhancer.

- True wake/sleep status indication
- Only light equipment required

Clinically validated

Highly reliable

PHYSIP
6, rue Gobert
75011 Paris, France
+33 1 42 17 00 10
aseega@physip.fr
www.physip.fr

PHYSIP

Physiological Signal Processing Research

Overview

- Develops ambulatory screening
- Increases diagnosed population
- Enhances apnea-hypopnea index (AHI)



Unsupervised wake/sleep scoring ●

Single channel analysis (EEG) ●


Based on 35 years of research ●

Applications

- Home polygraphy studies with EEG add-on
- Ambulatory screening for SDB
- Telemedicine

Aseega technology

- EEG signal from a single EEG lead conveys sufficient information for automatic wake/sleep scoring, without EOG and EMG information
- ASEEGA autoadaptive algorithm combines multiple advanced signal processing technologies based on time-scale analysis and fuzzy logic

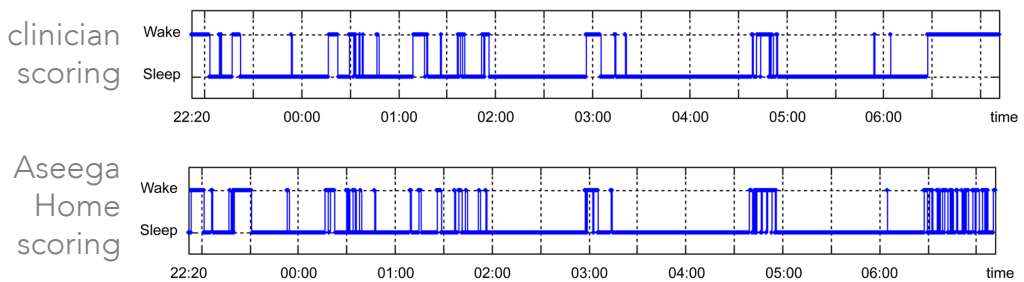

Sleep studies

- **Aseega now used in sleep research**
[Science 2009 Apr 24; 324 \(5926\):516-9](#)
 Part of this neuroscience study involved sleep scoring. Aseega, used as a fully automated scoring tool, enabled the exclusion of a bias resulting from possible visual scoring subjectivity
- **Clinical validation on healthy subjects**
 Berthomier et al. *Automatic Analysis of Single-Channel Sleep EEG: Validation in Healthy Individuals.* **Sleep 2007;30(11):1587-95**

| | | |
|--------------------|----------------------|--------|
| Scoring agreements | Wake / Sleep | : 96 % |
| | Wake / REM / NREM | : 92 % |
| | W / R / N1 / N2 / N3 | : 83 % |
- **Clinical validation on patients**

| | | |
|--------------------|-------------------|--------|
| Scoring agreements | Wake / Sleep | > 94 % |
| | Wake / REM / NREM | > 88 % |

Journal of Sleep Research 2006;15(Suppl 1):P295
- **Studies on large volunteer cohort and on patients *in situ* underway**

History

Aseega Technology is an extension of the work initiated by Odile Benoit and Jacques Prado (Le Roux 86, Bouard 86), who proposed a semi-automatic approach for the analysis of all-night sleep, based on a single EEG channel. At that time, this signal was recorded using a research prototype.

This semi-automatic method was routinely applied for 12 years by Benoit et al. to more than 500 recordings, including hypnotic pharmacological trials (Benoit 94, Daurat 97). Their work successfully demonstrated that scoring sleep from a single EEG channel is possible, although challenging.

This single channel method was used as a starting point for Christian Berthomier's PhD thesis in signal processing. His work, supervised by J. Prado in collaboration with Dr. O. Benoit, formed the basis of an automatic method to process sleep EEG recorded under laboratory conditions (Berthomier 99).

Since 2001, the research and development work carried out by the Physip Company, directed by C. Berthomier, has led to Aseega Technology. Aseega provides a fully automatic analysis of the EEG signal delivered by recording devices routinely used in sleep centers. The clinical validation of Aseega was successfully performed on 15 healthy subjects in 2005 and on 15 patients in 2006 (Berthomier 06, 07). The latest developments achieved real time scoring (Berthomier 08).

Aseega is now being used in sleep linked research domains (Schmidt 09, Van Beers 09).

Benoit O, Bouard G, Payan C, Borderies P and Prado J, *Effect of a single dose (10 mg) of zolpidem on visual and spectral analysis of sleep of poor young sleepers*, *Psychopharmacology*, 116, pp. 297-03, 1994.

Berthomier C, Prado J and Benoit O, *Automatic sleep EEG analysis using filter banks*, in *Biomedical Sciences Instrumentation*, 35(0), pp. 241-46, 1999.

Berthomier C, Drouot X, Herman-Stoïca M, Berthomier P, Prado J, Benoit O, Mattout J, d'Ortho M-P, *Wake-REM-NREM automatic classification based on a single EEG channel: epoch-by-epoch comparison with human sleep scoring in patients*. *Journal of Sleep Research*, Vol. 15/Suppl. 1, p168, 2006.

Berthomier C, Drouot X, Herman-Stoïca M, Berthomier P, Prado J, Bokar-thiré D, Benoit O, Mattout J, d'Ortho M-P, *Automatic analysis of single-channel sleep EEG: validation in healthy individuals*. *Sleep*, 30(11), pp. 1587-95, 2007.

Berthomier C, Herman-Stoïca M, Berthomier P, Drouot X, Prado J, Mattout J, d'Ortho MP, *Real-time automatic wake/sleep scoring based on a single EEG channel*. *Sleep*, Vol 31 (suppl.), A338, 2008

Bouard G, Benoit O, Lacombe J. *Real-time analysis and semi-automatic classification of sleep EEG signals using a microprocessor. Part II: Application to normal and disturbed sleep*. In: Court L, Trocherie S, Doucet J, eds. *Le traitement du signal en électrophysiologie expérimentale et clinique du système nerveux central*. Candé, France: Presses Lefrancq, 1986:531-44.

Daurat A, Aguirre A, Foret J and Benoit O, *Disruption of sleep recovery after 36h of exposure to moderately bright light*, *Sleep*, 20(5), pp. 352-58, 1997.

Le Roux J, Moreau N, Prado J. *Real-time analysis and semi-automatic classification of sleep EEG signals using a microprocessor. Part I: Principles of analysis*. In: Court L, Trocherie S, Doucet J, eds. *Le traitement du signal en électrophysiologie expérimentale et clinique du système nerveux central*. Candé, France: Presses Lefrancq, 1986:521-30.

Schmidt C, Collette F, Leclercq Y, Sterpenich V, Vandewalle G, Berthomier P, Berthomier C, Philipps C, Tinguely G, Darsaud A, Gais S, Schabus M, Desseilles M, Dang-Vu T, Salmon E, Balteau E, Degueldre C, Luxen A, Maquet P, Cajochen C, Peigneux P. *Homeostatic sleep pressure and responses to sustained attention in the suprachiasmatic area*. *Science*, 324 (5926):516-9, 2009.

Van Beers P, Berthomier C, Prado J, Berthomier P, Coste O. *HRV Analysis during Sleep: Clinical Interest of a New Automatic Sleep Analysis*. To be published in *Sleep Abstract Supplement*, Vol. 32, 2009.